

# PERFORMING THE ORAQUICK HIV SELF TEST

A TRAINING FLIP BOOK

# Before the test

---

*Do not eat, drink or use oral care products (mouthwash, toothpaste) 15 minutes before you start the test*

# This kit contains:




Preservative –  
Not needed for  
the test



Test stick




Test tube

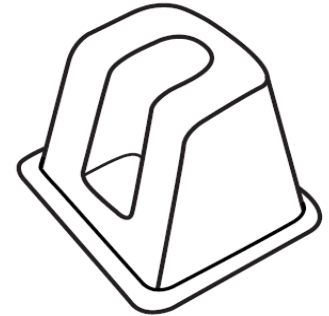


Tube holder

1

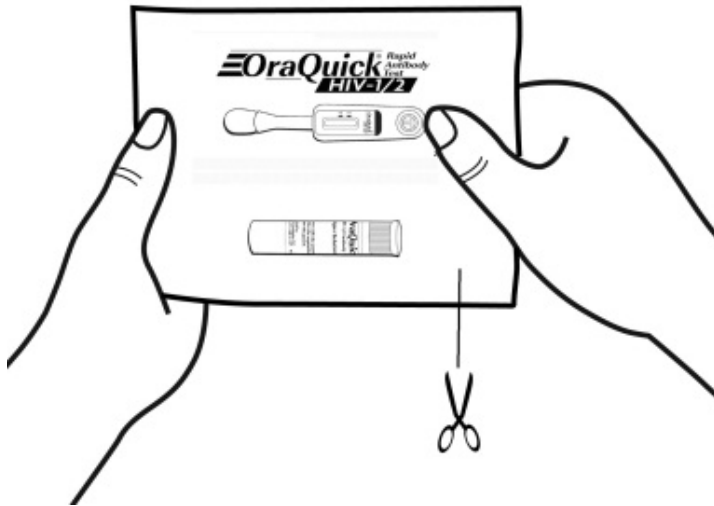

Tear open the packet

2

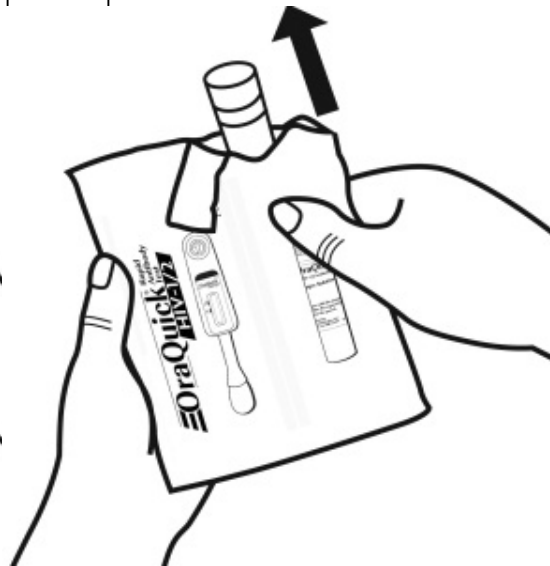

Locate and remove  
the test tube.

3




Remove the  
cap. Be sure to  
NOT spill the  
liquid inside.

4




Place tube in  
the tube holder

5

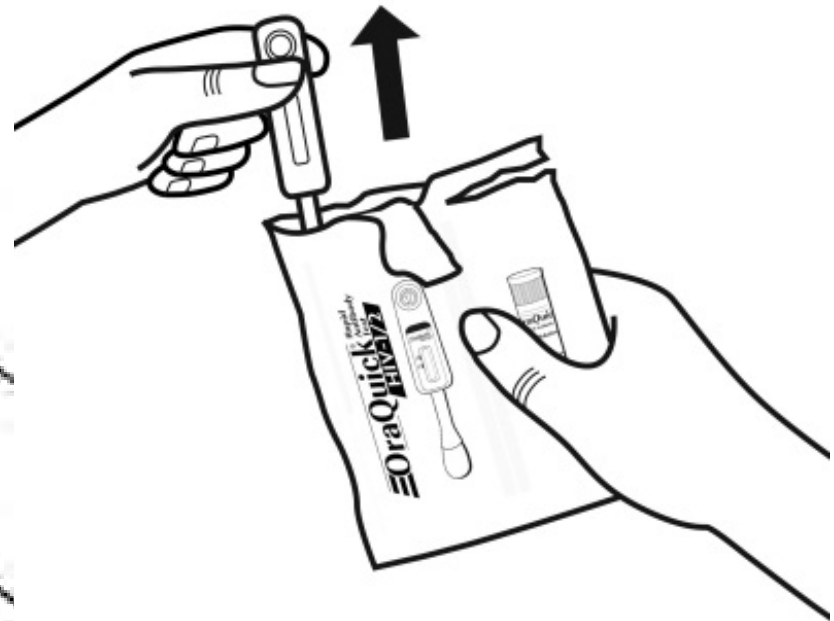


Locate and remove the  
test stick. Do not touch  
the pad with your fingers.

6

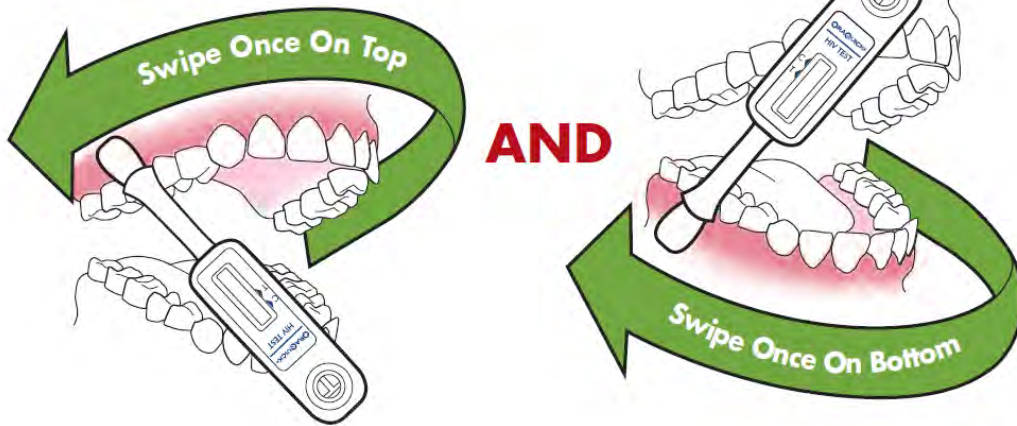


Swipe the pad along your upper gum once and your lower gum once.

7

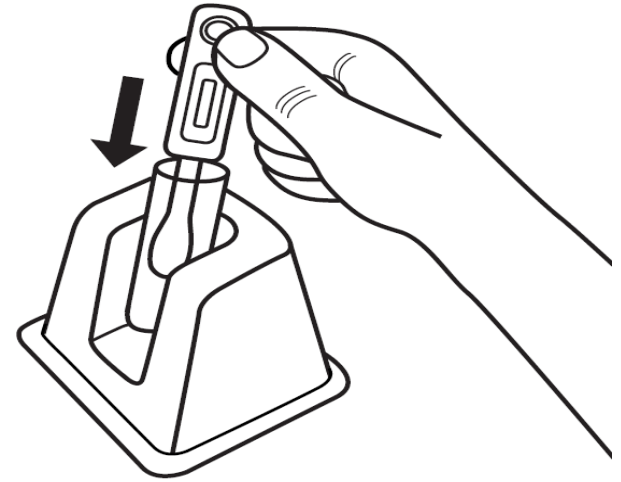

Place test stick into tube

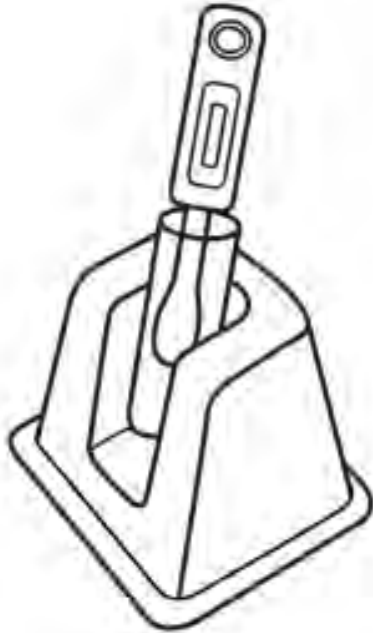

**20 min**

Wait 20 minutes. Do not read the results before 20 minutes or after 40 minutes have passed. You could get the wrong results


# Negative Result



If there is ONE LINE next to the “C” and NO line next to the “T” your test result is negative



# Positive Result

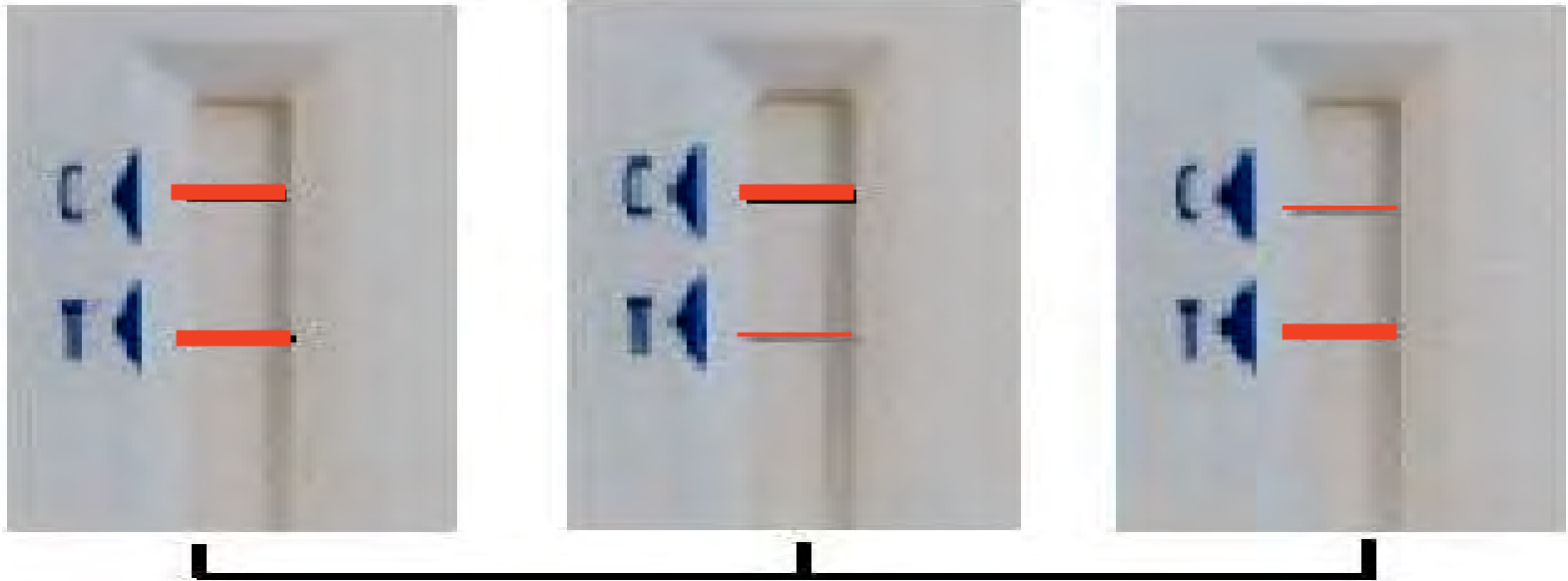

If there are TWO LINES, one next to the “C” and one next to the “T” – EVEN A VERY FAINT ONE you may have HIV

# Indeterminate Result



If there is no line next to the "C", if there is a line somewhere outside the "C" or "T" areas, or there are no lines, your test is NOT working

# Disposing of the test

- Remove the test stick, put the cap on the test tube and throw away all contents in the normal trash

*Review steps of the  
Oral Swab Testing SOP*